


Deep Conical (DC) 03.0mm Parallel Walled Implants - Drill Sequence

Ø3.0mm Parallel Walled (DCC30)




Soft Bone

D-3Spade
Ø2.0mm

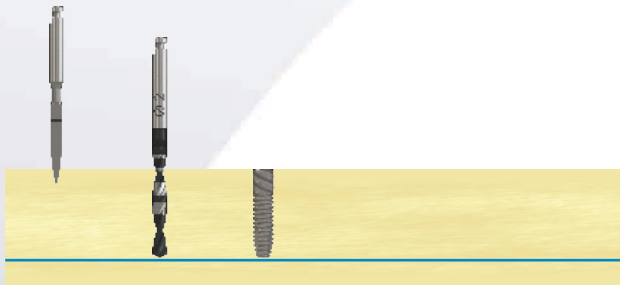


Medium Bone

D-3Spade
Ø2.0mm
Ø2.5mm

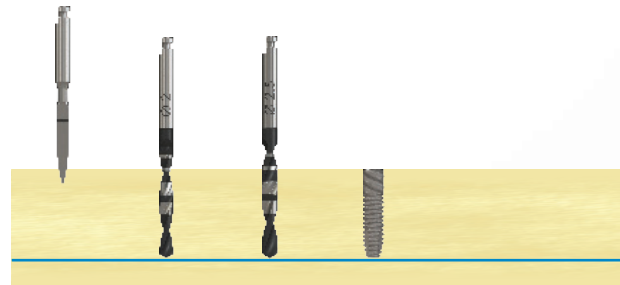

Dense Bone

D-3Spade
Ø2.0mm
Ø2.7mm

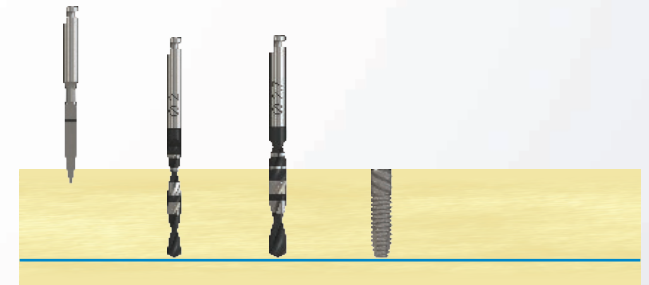

NOTE: Site preparation Sequence recommended by Southern Implants does not replace the judgement and experience of the surgeon.



SOUTHERNIMPLANTS

Innovative Treatment Solutions

www.southernimplants.com